




CHANGING THE “NEXT SAFETY CHECK” DATE

Step-by-Step process for VIO 300D ESU



1. Turn the VIO 300D on. Once the “*Safety Check Due*” (green message box) appears, press any tabs (“▶” Display Left-side, “◀” Display Right -side) keys to clear the message.
2. Select “Return” option.
3. Select “Other Functions” option.
4. Select “Setup” option.
5. Press the “More” “◀” option until the “Service Program” option is available, select this option.

The unit will now request a Password. The white buttons (▲ = Up, ▼ = Down and  = Confirm/Save), that are located to the right of the Front Display, will be used to enter the password.

6. Press the “▼” button once and observe that the “—” (hyphen/dash) now occupies the first character position of the password field.
7. Press the  “button, this will allow you to gain access to the “Setup” options.
8. Review the current “Date and Time” options; and if required, make the necessary corrections by simply selecting/highlighting the Date and/or Time option, while using the “▼” and “▲” buttons to adjust the settings.
9. Press the More “◀” option several times until the “Next Safety Check” option is available. Select/highlight the option.
10. Press the “▲” button once; confirm that the “Next Safety Check” date adjust to a year from the current date. Press the , and turn the unit off.
11. Turn the unit on and confirm that the green message box no longer appears.

NOTE: The green message will re-appear a year from this update.