

ACCESSING AND CHANGING THE PAD CONFIGURATION ON THE VIO SYSTEM



VIO PAD CONFIGURATION CHANGE

- From any program screen, (Basic, Program 1-10 etc)
- Select Other functions
- Select Set up
- Select More Twice
- Select Service program
- Touch Down Arrow once to “ – “
- Press Enter button
- Select Neutral electrode
- Change the Selection from Either way to Single surface
- Select Return twice, This will return you to whatever screen you started with
- Repeat process to return to original configuration