



With its highly flexible programming architecture and intuitive controls, ENDO CUT® I is an innovative approach to interventional endoscopy

Only VIO® offers the next generation in ENDO CUT® technology with proprietary voltage and spark control. With ENDO CUT® I, the physician is able to independently tailor cutting duration and interval. Additionally, the effect settings have the ability to select from four different levels of hemostasis.




Independent Adjustable Parameters

- 01 Effect setting adjusts hemostasis level
- 02 Cut duration adjusts length of cut
- 03 Cut interval adjusts time between cutting phases

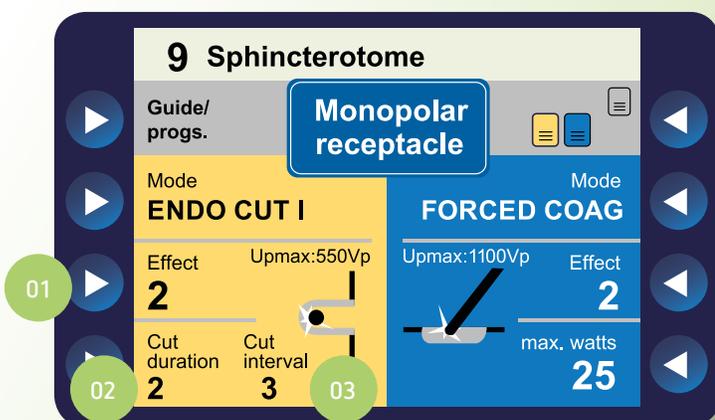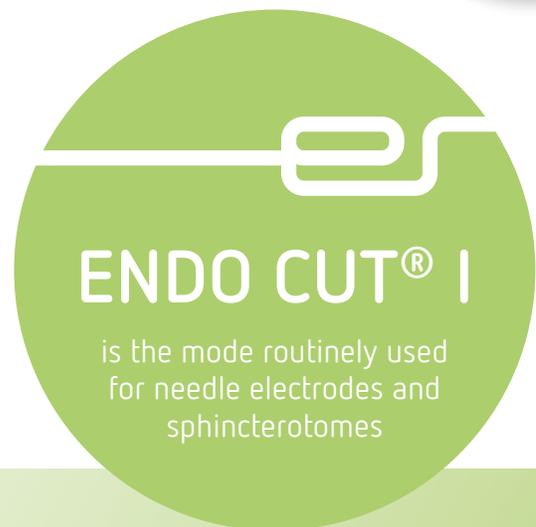


The coagulation phase allows for additional coagulation between the cutting phases. Coagulation intensity can be adjusted using the EFFECT setting, altering coagulation voltage levels. Raising the EFFECT setting increases coagulation intensity.

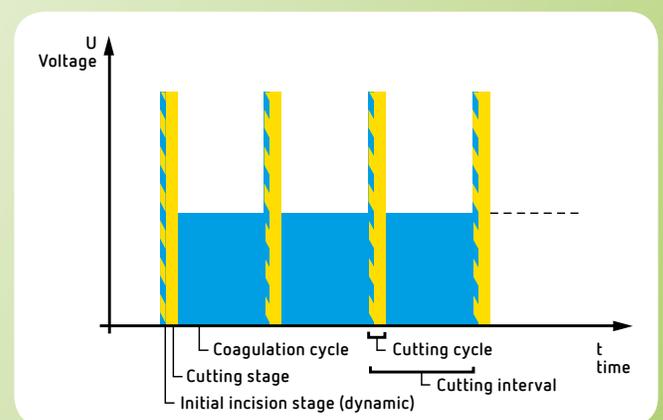

Cutting cycle with ENDO CUT® I:
initial incision stage (yellow/blue),
cutting stage (yellow) and coagulation cycle (blue).



With its highly flexible programming architecture and intuitive controls, ENDO CUT® Q is an innovative approach to interventional endoscopy

Only VIO® offers the next generation in ENDO CUT® technology with proprietary voltage and spark control. With ENDO CUT® Q, the physician is able to independently tailor cutting duration and interval. Additionally, the effect settings have the ability to select from four different levels of hemostasis.

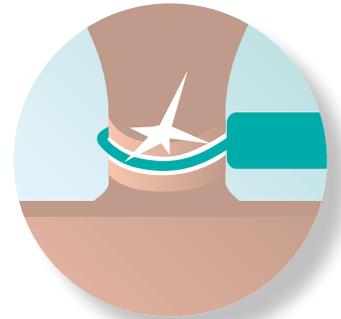


Independent Adjustable Parameters

- 01 Effect setting adjusts hemostasis level
- 02 Cut duration adjusts length of cut
- 03 Cut interval adjusts time between cutting phases

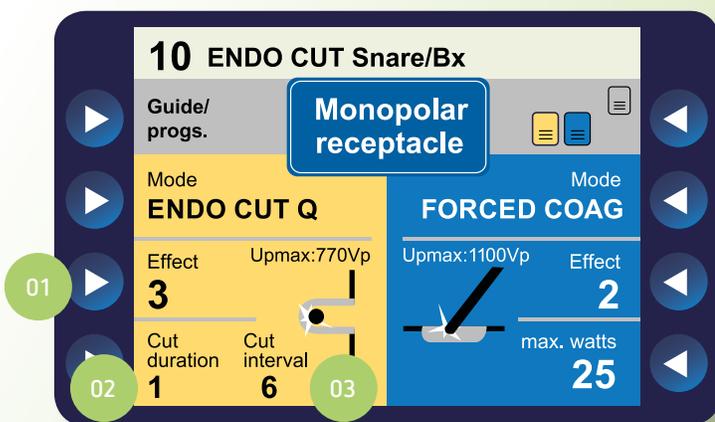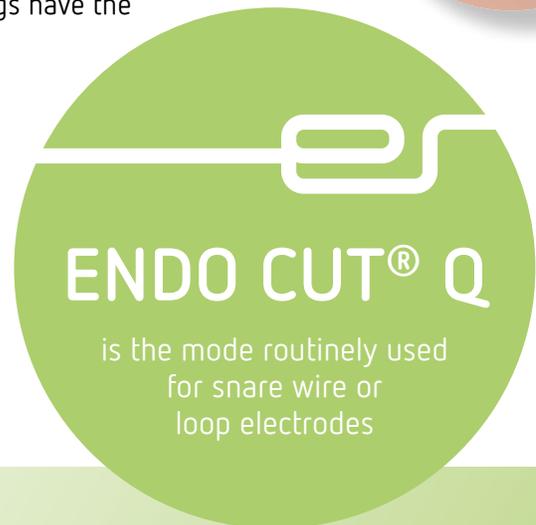

The coagulation phase allows for additional coagulation between the cutting phases. Coagulation intensity can be adjusted using the EFFECT setting, altering coagulation voltage levels. Raising the EFFECT setting increases coagulation intensity.

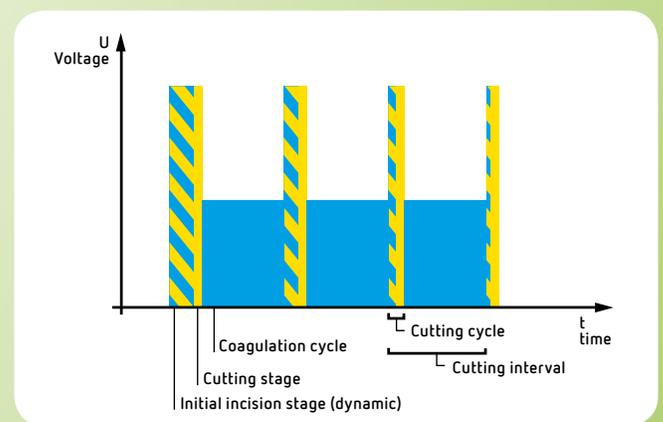

Cutting cycle with ENDO CUT® Q:
initial incision stage (yellow/blue),
cutting stage (yellow) and coagulation cycle (blue)

Erbe USA Incorporated
2225 Northwest Parkway
Marietta, GA 30067
USA

Phone 800 778 3723
Fax 770 955 2577
sales@erbe-usa.com
erbe-usa.com